


Cassette Wrapping and Taping Procedures

The RH-Pro11's "Wrapped Cassette" cycle allows the successful sterilization of instruments in paper-wrapped cassettes. To assure instrument sterilization after the sterilization cycle, proper procedures must be followed when preparing the cassette for sterilization. Use only paper wrap and sterilization tape sold through CPAC Equipment, Inc. to provide the heat resistance necessary to assure wrap and tape performance. The procedures provided below are those recommended by AAMI and AORN for wrapping instrument cassettes.


Step 1

Place cassette in the middle of the paper wrap aligning the corners of the wrap with the center point of each side of the cassette.

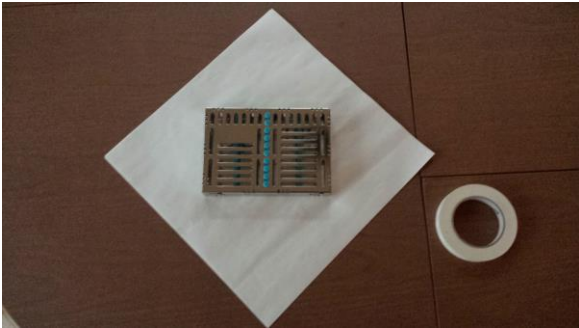


Step 2

Fold bottom wrap corner toward the top mid point of the cassette.

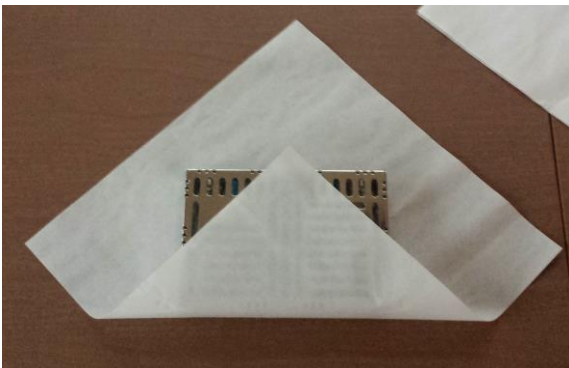


Step 3

Fold over corner back onto wrap and securely attach sterile tape by firmly pressing the tape to the wrap. Note that the tape is not attached to the cassette.

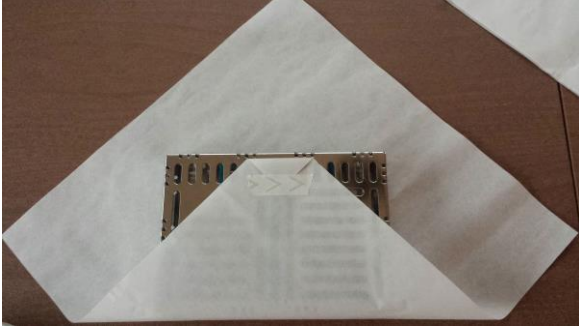


Step 4

Fold left side by pushing in bottom left corner and folding over remainder of flap until it is horizontally aligned with wrap's edges (Left). Push down lower left fold (Right)

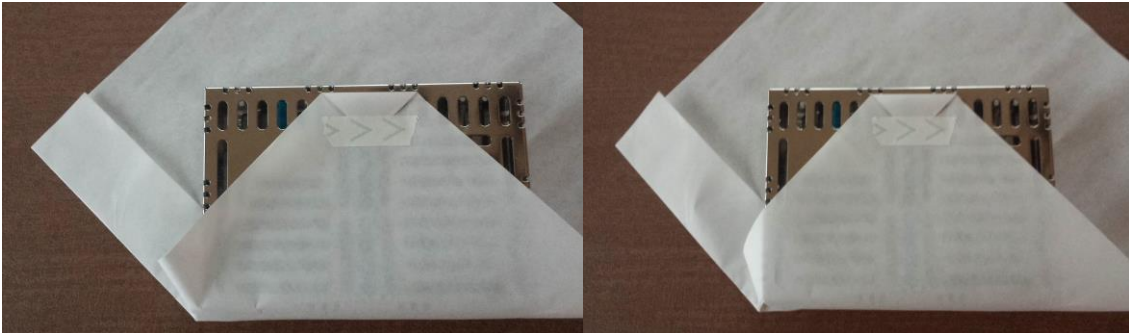



Step 5

Fold over left corner and tape securely to first-fold wrap.

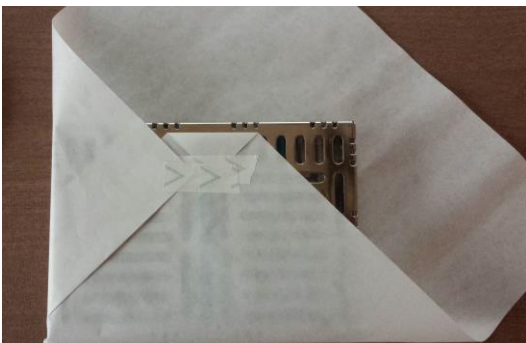


Step 6

Fold over right side of wrap, repeating Steps 4 and 5.

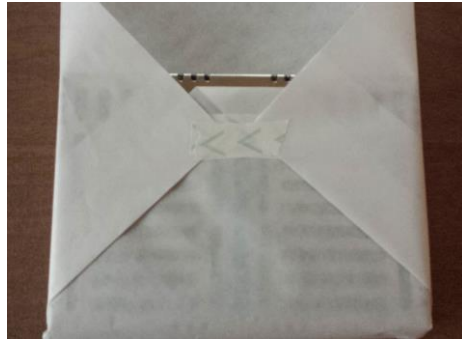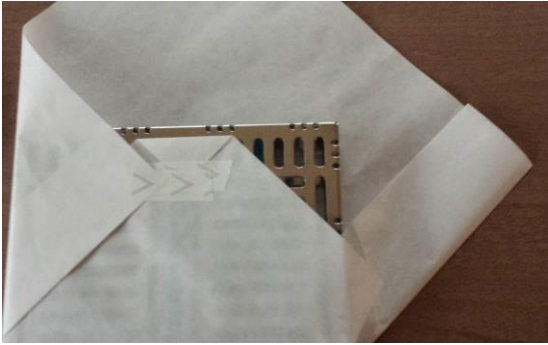

Step 7

Fold top by pushing in bottom left corner and folding over remainder of flap until it is horizontally aligned with wrap's edges (Left). Repeat for right side of top flap (Right).

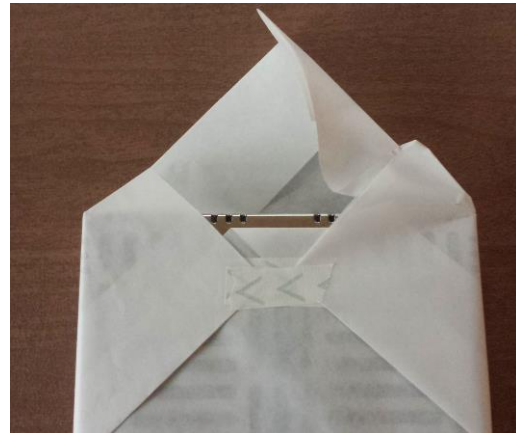

Step 8

Fold over right top flap to cover left top flap (Left) and turn down onto cassette and firmly tape.



Step 9

Repeat Steps 1-8 for double wrapping.

Step 10

Complete sterilization process.

